


Better guidance for more confident prostate biopsy decisions.

Access Hybritech p2PSA* and Prostate Health Index (ϕ i)


Blood Banking
Centrifugation
Chemistry
Flow Cytometry
Hematology
Hemostasis
Immunoassay
Information Systems
Lab Automation
Molecular Diagnostics
Rapid Diagnostics

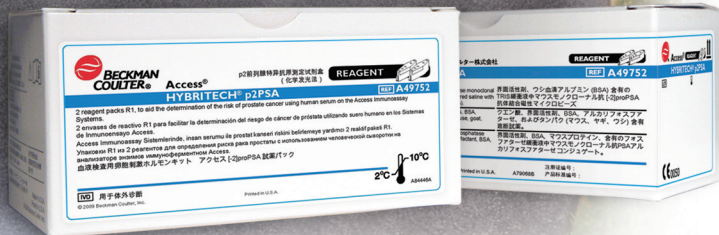

Several isoforms of free prostate-specific antigen (fPSA) have been identified as precursor forms of PSA (proPSA). One form, [-2]proPSA, has been determined to be the most specific for prostate cancer.¹ The new Access Hybritech p2PSA test measures the [-2]proPSA concentration in serum.

When p2PSA measurements are combined with Access Hybritech PSA and free PSA, the resulting index demonstrates a significant improvement in clinical specificity for prostate cancer detection, relative to PSA and fPSA alone in the PSA range of 2-10 ng/mL in men 50 years of age or older and with non-suspicious digital rectal exam (DRE) findings. This resulting index, a patient's personalized prostate cancer risk assessment, is known as the Prostate Health Index or ϕ i.^{2,3,4}

Prostate Cancer Detection: % Specificity of Beckman Coulter ϕ i Relative to PSA and % fPSA at 90% Clinical Sensitivity

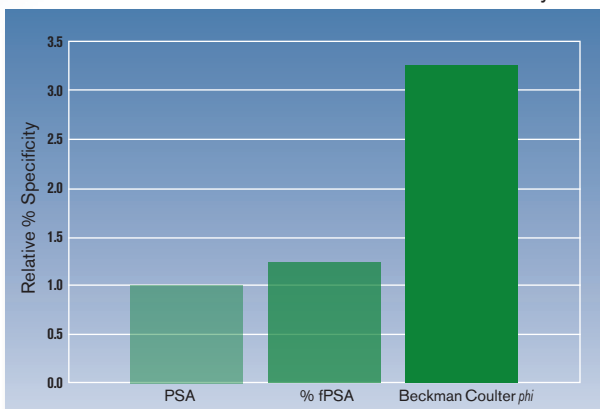

Beckman Coulter ϕ i offers:


- Enhanced clinical specificity – provides significantly improved clinical specificity leading to a potential reduction of unnecessary biopsies
- Personalized results – provides individualized risk assessment for prostate cancer
- Automated results - when Hybritech p2PSA is used in conjunction with Hybritech PSA and free PSA
- Equally effective - when used with either Hybritech or WHO calibrations for PSA and free PSA

Access Hybritech p2PSA and *phi*

The *phi* results provide patients more specific information about the prostate cancer risk their elevated PSA may represent. A low *phi* result indicates a potentially lower risk of prostate cancer, while an elevated *phi* result may suggest that a recommendation for a prostate biopsy is more necessary (see charts below).

Beckman Coulter *phi* can be used to categorize patients into appropriate low, moderate and high probabilities of prostate cancer found on biopsy. The resultant *phi* ranges have been associated with increasing probabilities of cancer as listed below:

Probability of Prostate Cancer in Patients with PSA levels in the 2-10 ng/mL range using <i>phi</i> with Hybritech calibrations			
<i>phi</i> Result Range (Hybritech Calibration)	Prostate Cancer Risk Category	Probability of Cancer	95% Confidence Interval
0-20.9	Low	8.4%	1.9% - 16.1%
21.0 - 39.9	Moderate	21.0%	17.3% - 24.6%
40+	High	44.0%	36.0% - 52.9%
Probability of Prostate Cancer in Patients with PSA levels in the 1.6-7.8 ng/mL range using <i>phi</i> with WHO calibrations			
<i>phi</i> Result Range (WHO Calibration)	Prostate Cancer Risk Category	Probability of Cancer	95% Confidence Interval
0-22.9	Low	8.7%	2.0% - 17.0%
23.0 - 44.9	Moderate	20.6%	17.1% - 24.1%
45.0+	High	43.8%	35.8% - 52.2%

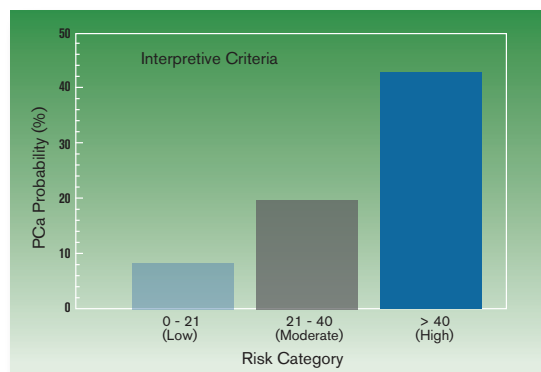

*In patients with tPSA between 2-10 ng/mL using Hybritech calibration

Characteristics

Sample Type	Serum
Time to First Result	Approx. 20 mins
Limit of Blank (LOB)	0.50 pg/mL
Dynamic Range	Approx. 0.50 to 5000 pg/mL
Open Pack Stability	28 days
Calibration Stability	28 days
Limit of Quantitation (LOQ)	3.23 pg/mL

Ordering Information

Reagent Kit (2 x 50 tests)	A49752
Calibrator Kit (7 x 2.1 mL)	A49753
Control Kit (3 levels; 1 x 5.0 mL)	A56934

References

1. Mikolajczyk SD. Proenzyme forms of prostate-specific antigen in serum improve detection of prostate cancer. Clin Chem 2004 March;50:1017-1025.
2. Jansen FH. Prostate-specific antigen (PSA) isoform p2PSA in combination with total PSA and free PSA improves diagnostic accuracy in prostate cancer detection. Eur Urol 2010;57(6):921-927.
3. Sokoll L. [-2]proenzyme prostate-specific antigen for prostate cancer detection: a National Cancer Institute Early Detection Research Network validation study. J Urol 2008;180(2):539-543.
4. Le BV. [-2]proenzyme prostate-specific antigen is more accurate than total and free prostate-specific antigen in differentiating prostate cancer from benign disease in prospective prostate cancer screening study. J Urol 2010;183:1355-59.

Access Tumor Markers: A Menu that Matters

Hybritech PSA
BR Monitor
CEA

Hybritech free PSA
GI Monitor
AFP

Hybritech p2PSA*
OV Monitor

*Not available in the U.S.

Access tumor markers are part of a comprehensive assay menu featured on the Access and UniCel Immunoassay systems. To learn more, visit www.beckmancoulter.com/tumormarkers.



Stay Connected

With your web-enabled camera phone scan the QR Code with a QR Code reader application. Get the latest information at

www.ProstateHealthIndex.org

Space for
FSC Logo

Beckman Coulter, the stylized logo, Access, Hybritech and UniCel are registered trademarks of Beckman Coulter, Inc.

Printed with soy ink



Australia, Gladesville (61) 2 9844 6000 Austria, Vienna (43) 662 857220 72 Canada, Mississauga (1) 905 819 1234
China, Shanghai (86) 21 3865 1000 Croatia, Zagreb (38) 51 489 9003 Czech Republic, Prague (420) 272 017 999
Eastern Europe, Middle East, North Africa, South West Asia: Switzerland, Nyon (41) 22 365 3707 France, Villepinte (33) 1 49 90 90 00
Germany, Krefeld (49) 2151 33 35 Hong Kong (852) 2814 7431 Hungary, Budapest (361) 25 09 330 India, Mumbai (91) 22 3080 5000
Italy, Cassina de' Pecchi, Milan (39) 02 953921 Japan, Tokyo (81) 3 5530 8500 Korea, Seoul (82) 2 404 2146 Latin America (1) (305) 380 4709
Malaysia, Kuala Lumpur (60) 3 5621 4793 Mexico, Mexico City (001) 52 55 9183 2800 Netherlands, Woerden (31) 348 462462 Poland, Warszawa (48) 22 366 0180
Puerto Rico (787) 747 3335 Russia, Moscow (7) 495 9846730 Singapore (65) 6339 3633 South Africa/Sub-Saharan Africa, Johannesburg (27) 11 564 3203
Spain, Madrid (34) 91 3836080 Sweden, Bromma (46) 8 564 85 900 Switzerland, Nyon (41) 0800 850 810 Taiwan, Taipei (886) 2 2730 2500
Turkey, Istanbul (90) 216 570 17 17 UK, High Wycombe (44) 01494 441181 USA, Brea, CA (1) 800 352 3433, (1) 714 993 5321

B2010-11215 DG-2K

www.beckmancoulter.com

© 2010 Beckman Coulter, Inc.

DIAG-PRINTED IN U.S.A.